

Table of contents

\\USER

Khan

NeuroAnalytics

DBS_3t

AAHead_Scout
T1w_MPR**
t2_space_dark-fluid_sag_p2_iso
diff_mb3_95dir_b2000_AP
diff_mb3_b0_PA
T2w_SPC
ep_bold_mb4_p2_2.5iso_rest
gre_field_mapping
HIPPO_DTI_1mm
HIPPO_DTI_1mm_AP
HIPPO_DTI_1mm_PA

\\USER\Khan\NeuroAnalytics\DBS_3t\AAHead_Scout

TA: 0:14 PM: REF Voxel size: 1.6×1.6×1.6 mmPAT: 3 Rel. SNR: 1.00 : fl

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	On
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	20 %
Position	L0.0 A10.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	128
FoV read	260 mm
FoV phase	100.0 %
Slice thickness	1.6 mm
TR	3.15 ms
TE	1.37 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	3.15 ms
TE	1.37 ms
Flip angle	8 deg

Contrast - Dynamic

Averages	1
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1

Resolution - Common

FoV read	260 mm
FoV phase	100.0 %
Slice thickness	1.6 mm
Base resolution	160
Phase resolution	100 %
Slice resolution	69 %
Phase partial Fourier	6/8
Slice partial Fourier	6/8
Trajectory	Cartesian

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	3
Ref. lines PE	24
Accel. factor 3D	1

Resolution - iPAT

Reference scan mode	Integrated
---------------------	------------

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	20 %
Position	L0.0 A10.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	0.0 %
Slices per slab	128
FoV read	260 mm
FoV phase	100.0 %
Slice thickness	1.6 mm
TR	3.15 ms
Multi-slice mode	Sequential
Series	Ascending
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	L0.0 A10.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off

System - Miscellaneous

Coil Select Mode	On - AutoCoilSelect
------------------	---------------------

System - Adjustments

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Non-sel.

System - Tx/Rx

Frequency 1H	123.219489 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Flip angle	8 deg
Measurements	1
Time to center	6.2 s

Inline - Inline

Subtract	Off
Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Inline - MapIt

Save original images	On
MapIt	None
Flip angle	8 deg
Measurements	1

Inline - MapIt

Contrasts	1
TR	3.15 ms
TE	1.37 ms

Sequence - Part 1

Introduction	On
Dimension	3D
Asymmetric echo	Weak
Contrasts	1
Multi-slice mode	Sequential
Bandwidth	540 Hz/Px

Sequence - Part 2

RF pulse type	Fast
Gradient mode	Normal
Excitation	Non-sel.
RF spoiling	On

Sequence - Assistant

Mode	Off
------	-----

\\USER\Khan\NeuroAnalytics\DBS_3t\T1w_MPR**

TA: 6:38 PM: FIX Voxel size: 0.8×0.8×0.8 mmPAT: 2 Rel. SNR: 1.00 : tfi

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	50 %
Position	L0.0 P0.0 F35.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
Slice oversampling	25.0 %
Slices per slab	192
FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
TR	2400.0 ms
TE	2.28 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	2400.0 ms
TE	2.28 ms
Magn. preparation	Non-sel. IR
T1	1060 ms
Flip angle	8 deg
Fat suppr.	Water excit. fast
Water suppr.	None

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
Base resolution	320
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off
Slice partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	50 %
Position	L0.0 P0.0 F35.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	25.0 %
Slices per slab	192
FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
TR	2400.0 ms
Multi-slice mode	Single shot
Series	Ascending
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	L0.0 P0.0 F35.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm

System - Miscellaneous

MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Default

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	L0.0 P0.0 F35.0 mm
Orientation	Sagittal
Rotation	0.00 deg
A >> P	240 mm
F >> H	256 mm
R >> L	154 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Non-sel.

System - Tx/Rx

Frequency 1H	123.219489 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	2400.0 ms
Concatenations	1

Physio - Cardiac

Magn. preparation	Non-sel. IR
TI	1060 ms
Fat suppr.	Water excit. fast
Dark blood	Off
FoV read	256 mm
FoV phase	93.8 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Inline - MapIt

Save original images	On
MapIt	None
Flip angle	8 deg
Measurements	1
TR	2400.0 ms
TE	2.28 ms

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Reordering	Linear
Asymmetric echo	Allowed
Flow comp.	No
Multi-slice mode	Single shot
Echo spacing	8.5 ms
Bandwidth	210 Hz/Px

Sequence - Part 2

RF pulse type	Fast
Gradient mode	Performance
Excitation	Non-sel.
RF spoiling	On
Incr. Gradient spoiling	On
Turbo factor	240

Sequence - Assistant

Mode	Off
------	-----

\\USER\Khan\NeuroAnalytics\DBS_3f\l2_space_dark-fluid_sag_p2_iso

TA: 8:17 PM: FIX Voxel size: 0.8×0.8×0.8 mmPAT: 2 Rel. SNR: 1.00 : spcir

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	192
FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
TR	5000 ms
TE	387 ms
Averages	1.0
Concatenations	1
Filter	Raw filter, Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	5000 ms
TE	387 ms
MTC	Off
Magn. preparation	Non-sel. T2-IR
TI 1	1800 ms
Fat suppr.	None
Blood suppr.	Off
Restore magn.	Off

Contrast - Dynamic

Averages	1.0
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
Base resolution	320
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Allowed
Slice partial Fourier	7/8
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	24
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	On
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	0.0 %
Slices per slab	192
FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
TR	5000 ms
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

Geometry - Saturation

Fat suppr.	None
Restore magn.	Off
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Sagittal
Rotation	0.00 deg
A >> P	240 mm
F >> H	256 mm
R >> L	154 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Non-sel.

System - Tx/Rx

Frequency 1H	123.219489 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
Trigger delay	0 ms
TR	5000 ms
Concatenations	1

Physio - Cardiac

Magn. preparation	Non-sel. T2-IR
TI 1	1800 ms
Fat suppr.	None
Dark blood	Off
FoV read	256 mm
FoV phase	93.8 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Reordering	Linear
Flow comp.	No
Echo spacing	3.86 ms
Adiabatic-mode	Off
Bandwidth	744 Hz/Px

Sequence - Part 2

Echo train duration	919 ms
RF pulse type	Normal
Gradient mode	Fast
Excitation	Non-sel.
Flip angle mode	T2 var
Turbo factor	278

Sequence - Assistant

Allowed delay	30 s
---------------	------

\\USER\Khan\NeuroAnalytics\DBS_3\diff_mb3_95dir_b2000_AP

TA: 9:10 PM: FIX Voxel size: 1.6×1.6×1.6 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	90
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	224 mm
FoV phase	100.0 %
Slice thickness	1.60 mm
TR	5500 ms
TE	99.20 ms
Multi-band accel. factor	3
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	5500 ms
TE	99.20 ms
MTC	Off
Magn. preparation	None
Flip angle	90 deg
Refocus flip angle	160 deg
Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Enabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	224 mm
FoV phase	100.0 %
Slice thickness	1.60 mm
Base resolution	140
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Dynamic Field Corr.	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	90
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	224 mm
FoV phase	100.0 %
Slice thickness	1.60 mm
TR	5500 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	3

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Enabled
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain

System - Miscellaneous

Coil Select Mode	Default
------------------	---------

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	224 mm
R >> L	224 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.219489 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	5500 ms
Multi-band accel. factor	3

Physio - PACE

Resp. control	Off
Multi-band accel. factor	3

Diff - Neuro

Diffusion mode	Free
Diff. directions	95
Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm ²
b-value 2	2000 s/mm ²
b-value 1	1
b-value 2	1
Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
FA maps	On
Mosaic	On
Tensor	On
Noise level	50

Diff - Body

Diffusion mode	Free
Diff. directions	95
Diffusion Scheme	Monopolar
Diff. weightings	2

Diff - Body

b-value 1	0 s/mm ²
b-value 2	2000 s/mm ²
b-value 1	1
b-value 2	1
Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
Exponential ADC Maps	Off
FA maps	On
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm ²
Noise level	50

Diff - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	Off
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.93 ms
Bandwidth	1488 Hz/Px

Sequence - Part 2

EPI factor	140
Gradient mode	Performance*
Excitation	Standard
RF spoiling	Off

Sequence - Special

Excite pulse duration	3840 us
Refocus pulse duration	6400 us
Single-band images	Off
MB LeakBlock kernel	Off
MB dual kernel	Off
MB RF phase scramble	Off
Time-shifted MB RF	Off
SENSE1 coil combine	Off
Invert RO/PE polarity	Off
PF omits higher k-space	Off
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
Physio recording	Off

\\USER\Khan\NeuroAnalytics\DBS_3t\diff_mb3_b0_PA

TA: 0:28 PM: FIX Voxel size: 1.6×1.6×1.6 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	90
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	P >> A
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	224 mm
FoV phase	100.0 %
Slice thickness	1.60 mm
TR	5500 ms
TE	99.20 ms
Multi-band accel. factor	3
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	5500 ms
TE	99.20 ms
MTC	Off
Magn. preparation	None
Flip angle	90 deg
Refocus flip angle	160 deg
Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Enabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	224 mm
FoV phase	100.0 %
Slice thickness	1.60 mm
Base resolution	140
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Dynamic Field Corr.	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	90
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	P >> A
FoV read	224 mm
FoV phase	100.0 %
Slice thickness	1.60 mm
TR	5500 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	3

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	P >> A
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	-180.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Enabled
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain

System - Miscellaneous

Coil Select Mode	Default
------------------	---------

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	-180.00 deg
A >> P	224 mm
R >> L	224 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.219489 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	5500 ms
Multi-band accel. factor	3

Physio - PACE

Resp. control	Off
Multi-band accel. factor	3

Diff - Neuro

Diffusion mode	Free
Diff. directions	95
Diffusion Scheme	Monopolar
Diff. weightings	1
b-value	0 s/mm ²
b-value	1
Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
FA maps	Off
Mosaic	Off
Tensor	Off
Noise level	50

Diff - Body

Diffusion mode	Free
Diff. directions	95
Diffusion Scheme	Monopolar
Diff. weightings	1
b-value	0 s/mm ²
b-value	1

Diff - Body

Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
Exponential ADC Maps	Off
FA maps	Off
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm ²
Noise level	50

Diff - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	Off
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.93 ms
Bandwidth	1488 Hz/Px

Sequence - Part 2

EPI factor	140
Gradient mode	Performance*
Excitation	Standard
RF spoiling	Off

Sequence - Special

Excite pulse duration	3840 us
Refocus pulse duration	6400 us
Single-band images	Off
MB LeakBlock kernel	Off
MB dual kernel	Off
MB RF phase scramble	Off
Time-shifted MB RF	Off
SENSE1 coil combine	Off
Invert RO/PE polarity	Off
PF omits higher k-space	Off
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
Physio recording	Off

\\USER\Khan\NeuroAnalytics\DBS_3t\T2w_SPC

TA: 5:57 PM: FIX Voxel size: 0.8x0.8x0.8 mmPAT: 2 Rel. SNR: 1.00 : spcR

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	208
FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
TR	3200 ms
TE	564 ms
Averages	1.0
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	3200 ms
TE	564 ms
MTC	Off
Magn. preparation	None
Fat suppr.	None
Blood suppr.	Off
Restore magn.	On

Contrast - Dynamic

Averages	1.0
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
Base resolution	320
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Allowed
Slice partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	0.0 %
Slices per slab	208
FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
TR	3200 ms
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

Geometry - Saturation

Fat suppr.	None
Restore magn.	On
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - AutoCoilSelect

System - Adjustments

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Non-sel.

System - Tx/Rx

Frequency 1H	123.219489 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
Trigger delay	0 ms
TR	3200 ms
Concatenations	1

Physio - Cardiac

Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	256 mm
FoV phase	93.8 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Subtract	Off
----------	-----

Inline - Common

Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Reordering	Linear
Flow comp.	No
Echo spacing	3.86 ms
Adiabatic-mode	Off
Bandwidth	744 Hz/Px

Sequence - Part 2

Echo train duration	1173 ms
RF pulse type	Fast
Gradient mode	Fast
Excitation	Non-sel.
Flip angle mode	T2 var
Turbo factor	314

Sequence - Assistant

Allowed delay	30 s
---------------	------

\\USER\Khan\NeuroAnalytics\DBS_3tlep_bold_mb4_p2_2.5iso_rest

TA: 8:17 PM: FIX Voxel size: 2.5×2.5×2.5 mmPAT: 2 Rel. SNR: 1.00 : epfid

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	220 mm
FoV phase	100.0 %
Slice thickness	2.50 mm
TR	1250 ms
TE	39.00 ms
Multi-band accel. factor	4
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	1250 ms
TE	39.00 ms
MTC	Off
Magn. preparation	None
Flip angle	50 deg
Fat suppr.	Fat sat.

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	384
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	220 mm
FoV phase	100.0 %
Slice thickness	2.50 mm
Base resolution	88
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	24
Reference scan mode	FLEET

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	220 mm
FoV phase	100.0 %
Slice thickness	2.50 mm
TR	1250 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	4

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	On - AutoCoilSelect

System - Adjustments

B0 Shim mode	Brain
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	220 mm
R >> L	220 mm
F >> H	150 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.219489 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	1250 ms
Multi-band accel. factor	4

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	20
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Baseline
Meas[4]	Baseline
Meas[5]	Baseline
Meas[6]	Baseline
Meas[7]	Baseline
Meas[8]	Baseline
Meas[9]	Baseline
Meas[10]	Baseline
Meas[11]	Active
Meas[12]	Active
Meas[13]	Active
Meas[14]	Active
Meas[15]	Active
Meas[16]	Active
Meas[17]	Active
Meas[18]	Active
Meas[19]	Active
Meas[20]	Active
Motion correction	Off

BOLD

Spatial filter	Off
Measurements	384
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.65 ms
Bandwidth	1832 Hz/Px

Sequence - Part 2

EPI factor	88
Gradient mode	Performance
Excitation	Standard
RF spoiling	Off

Sequence - Special

Excite pulse duration	3200 us
FLEET ref. prep. scans	0
FLEET ref. min. TR	0 ms
Single-band images	Off
MB LeakBlock kernel	Off
MB dual kernel	Off
MB RF phase scramble	Off
SENSE1 coil combine	Off
Invert RO/PE polarity	Off
PF omits higher k-space	Off
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
FLEET iPAT ref. FA	12.0 deg
Physio recording	DICOM
Triggering scheme	Standard

\\USER\Khan\NeuroAnalytics\DBS_3\gre_field_mapping

TA: 1:23 PM: REF Voxel size: 3.0×3.0×3.0 mmRel. SNR: 1.00 : fm_r

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	46
Dist. factor	25 %
Position	L0.0 A18.8 F18.4 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	3.0 mm
TR	500.0 ms
TE 1	4.92 ms
TE 2	7.38 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	HEA;HEP

Contrast - Common

TR	500.0 ms
TE 1	4.92 ms
TE 2	7.38 ms
MTC	Off
Flip angle	60 deg
Fat suppr.	None

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magn./Phase
Measurements	1
Multiple series	Off

Resolution - Common

FoV read	240 mm
FoV phase	100.0 %
Slice thickness	3.0 mm
Base resolution	80
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off

Resolution - Filter Image

Prescan Normalize	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	46
Dist. factor	25 %
Position	L0.0 A18.8 F18.4 mm
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	3.0 mm
TR	500.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	L0.0 A18.8 F18.4 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 A18.8 F18.4
L	0.0 mm
A	18.8 mm
F	18.4 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	None
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	On - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	L0.0 A18.8 F18.4 mm
Orientation	Transversal
Rotation	0.00 deg
A >> P	240 mm
R >> L	240 mm
F >> H	172 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
--------------	----------

System - Tx/Rx

Frequency 1H	123.219489 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Sequence - Part 1

Introduction	On
Dimension	2D
Asymmetric echo	Off
Contrasts	2
Flow comp.	Yes
Multi-slice mode	Interleaved
Bandwidth	291 Hz/Px

Sequence - Part 2

RF pulse type	Normal
Gradient mode	Fast
RF spoiling	On

Sequence - Assistant

Mode	Off
------	-----

\\USER\Khan\NeuroAnalytics\DBS_3t\HIPPO_DTI_1mm

TA: 6:27 PM: FIX Voxel size: 1.0×1.0×1.0 mmPAT: 2 Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	20
Dist. factor	0 %
Position	L0.0 A18.0 H0.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Temporal lobe
Phase oversampling	0 %
FoV read	220 mm
FoV phase	98.2 %
Slice thickness	1.0 mm
TR	2900 ms
TE	72.0 ms
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	2900 ms
TE	72.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	Fat sat.
Fat sat. mode	Strong

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	220 mm
FoV phase	98.2 %
Slice thickness	1.0 mm
Base resolution	220
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

Accel. mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	38
Reference scan mode	EPI/separate

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Dynamic Field Corr.	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	20
Dist. factor	0 %
Position	L0.0 A18.0 H0.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	220 mm
FoV phase	98.2 %
Slice thickness	1.0 mm
TR	2900 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	L0.0 A18.0 H0.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Temporal lobe
Initial Position	L0.0 A18.0 H0.0
R	0.0 mm
A	18.0 mm
F	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Fat sat. mode	Strong
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Matrix Optimization	Off
AutoAlign	Head > Temporal lobe

System - Miscellaneous

Coil Select Mode	Default
------------------	---------

System - Adjustments

B0 Shim mode	Advanced
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	L0.0 A18.0 H0.0 mm
Orientation	Transversal
Rotation	0.00 deg
A >> P	216 mm
R >> L	220 mm
F >> H	20 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.219489 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	2900 ms
Concatenations	1

Physio - PACE

Resp. control	Off
Concatenations	1

Diff - Neuro

Diffusion mode	MDDW
Diff. directions	64
Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm ²
b-value 2	500 s/mm ²
b-value 1	2
b-value 2	2
Diff. weighted images	On
Trace weighted images	On
ADC maps	On
FA maps	On
Mosaic	On
Tensor	On
Noise level	0

Diff - Body

Diffusion mode	MDDW
Diff. directions	64
Diffusion Scheme	Monopolar
Diff. weightings	2

Diff - Body

b-value 1	0 s/mm ²
b-value 2	500 s/mm ²
b-value 1	2
b-value 2	2
Diff. weighted images	On
Trace weighted images	On
ADC maps	On
Exponential ADC Maps	Off
FA maps	On
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm ²
Noise level	0

Diff - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	On
Optimization	Min. TE
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.93 ms
Bandwidth	1420 Hz/Px

Sequence - Part 2

EPI factor	216
RF pulse type	Normal
Gradient mode	Performance
Excitation	Standard

Sequence - pTX Pulses

\\USER\Khan\NeuroAnalytics\DBS_3t\HIPPO_DTI_1mm_AP

TA: 3:30 PM: FIX Voxel size: 1.0×1.0×1.0 mmPAT: 2 Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	20
Dist. factor	0 %
Position	R1.7 A18.0 F49.0 mm
Orientation	T > C17.5 > S-4.4
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	220 mm
FoV phase	98.2 %
Slice thickness	1.0 mm
TR	2900 ms
TE	74.0 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	2900 ms
TE	74.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	Fat sat.
Fat sat. mode	Strong

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	220 mm
FoV phase	98.2 %
Slice thickness	1.0 mm
Base resolution	220
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

Accel. mode	GRAPPA
Accel. factor PE	2

Resolution - iPAT

Ref. lines PE	38
Reference scan mode	EPI/separate

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Dynamic Field Corr.	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	20
Dist. factor	0 %
Position	R1.7 A18.0 F49.0 mm
Orientation	T > C17.5 > S-4.4
Phase enc. dir.	A >> P
FoV read	220 mm
FoV phase	98.2 %
Slice thickness	1.0 mm
TR	2900 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	R1.7 A18.0 F49.0 mm
Orientation	T > C17.5 > S-4.4
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	R1.7 A18.0 F49.0
R	1.7 mm
A	18.0 mm
F	49.0 mm
Initial Rotation	0.00 deg
Initial Orientation	T > C
T > C	17.5
> S	-4.4

Geometry - Saturation

Fat suppr.	Fat sat.
Fat sat. mode	Strong
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm

System - Miscellaneous

MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - AutoCoilSelect

System - Adjustments

B0 Shim mode	Advanced
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	R1.7 A18.0 F49.0 mm
Orientation	T > C17.5 > S-4.4
Rotation	0.00 deg
A >> P	216 mm
R >> L	220 mm
F >> H	20 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.219489 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	2900 ms
Concatenations	1

Physio - PACE

Resp. control	Off
Concatenations	1

Diff - Neuro

Diffusion mode	Free
Diff. directions	68
Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm ²
b-value 2	500 s/mm ²
b-value 1	1
b-value 2	1
Diff. weighted images	On
Trace weighted images	On
ADC maps	On
FA maps	On
Mosaic	On
Tensor	On

Diff - Neuro

Noise level	0
-------------	---

Diff - Body

Diffusion mode	Free
Diff. directions	68
Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm ²
b-value 2	500 s/mm ²
b-value 1	1
b-value 2	1
Diff. weighted images	On
Trace weighted images	On
ADC maps	On
Exponential ADC Maps	Off
FA maps	On
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm ²
Noise level	0

Diff - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	On
Optimization	Min. TE
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.93 ms
Bandwidth	1420 Hz/Px

Sequence - Part 2

EPI factor	216
RF pulse type	Normal
Gradient mode	Performance
Excitation	Standard

Sequence - pTX Pulses

\\USER\Khan\NeuroAnalytics\DBS_3\HIPPO_DTI_1mm_PA

TA: 3:30 PM: FIX Voxel size: 1.0×1.0×1.0 mmPAT: 2 Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	20
Dist. factor	0 %
Position	R1.7 A18.0 F49.0 mm
Orientation	T > C17.5 > S-4.4
Phase enc. dir.	P >> A
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	220 mm
FoV phase	98.2 %
Slice thickness	1.0 mm
TR	2900 ms
TE	74.0 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	2900 ms
TE	74.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	Fat sat.
Fat sat. mode	Strong

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	220 mm
FoV phase	98.2 %
Slice thickness	1.0 mm
Base resolution	220
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

Accel. mode	GRAPPA
Accel. factor PE	2

Resolution - iPAT

Ref. lines PE	38
Reference scan mode	EPI/separate

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Dynamic Field Corr.	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	20
Dist. factor	0 %
Position	R1.7 A18.0 F49.0 mm
Orientation	T > C17.5 > S-4.4
Phase enc. dir.	P >> A
FoV read	220 mm
FoV phase	98.2 %
Slice thickness	1.0 mm
TR	2900 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	R1.7 A18.0 F49.0 mm
Orientation	T > C17.5 > S-4.4
Phase enc. dir.	P >> A
AutoAlign	Head > Brain
Initial Position	R1.7 A18.0 F49.0
R	1.7 mm
A	18.0 mm
F	49.0 mm
Initial Rotation	180.00 deg
Initial Orientation	T > C
T > C	17.5
> S	-4.4

Geometry - Saturation

Fat suppr.	Fat sat.
Fat sat. mode	Strong
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm

System - Miscellaneous

MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - AutoCoilSelect

System - Adjustments

B0 Shim mode	Advanced
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	R1.7 A18.0 F49.0 mm
Orientation	T > C17.5 > S-4.4
Rotation	180.00 deg
A >> P	216 mm
R >> L	220 mm
F >> H	20 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.219489 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	2900 ms
Concatenations	1

Physio - PACE

Resp. control	Off
Concatenations	1

Diff - Neuro

Diffusion mode	Free
Diff. directions	68
Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm ²
b-value 2	500 s/mm ²
b-value 1	1
b-value 2	1
Diff. weighted images	On
Trace weighted images	On
ADC maps	On
FA maps	On
Mosaic	On
Tensor	On

Diff - Neuro

Noise level	0
-------------	---

Diff - Body

Diffusion mode	Free
Diff. directions	68
Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm ²
b-value 2	500 s/mm ²
b-value 1	1
b-value 2	1
Diff. weighted images	On
Trace weighted images	On
ADC maps	On
Exponential ADC Maps	Off
FA maps	On
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm ²
Noise level	0

Diff - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	On
Optimization	Min. TE
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.93 ms
Bandwidth	1420 Hz/Px

Sequence - Part 2

EPI factor	216
RF pulse type	Normal
Gradient mode	Performance
Excitation	Standard

Sequence - pTX Pulses